


SMI-SPON

ABSORBABLE GELATIN SPONGE

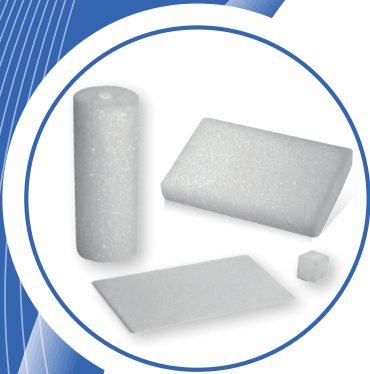


Product highlights

- ✓ Secure and quick haemostasis, stops bleeding effectively - Significantly reduces blood loss
- ✓ Absorbs 30-40 times its own weight in blood and fluids
- ✓ Can easily be reduced in size to any required format- does not stick to gloves or instruments
- ✓ May be applied dry or be soaked with sterile physiological solution or antibiotics
- ✓ Gamma sterilized, individually blister packed and ready to use

Benefits

- ✓ No risk of encapsulation
- ✓ Does not disintegrate after moistening
- ✓ Fasten recovery time for the patient
- ✓ Biocompatible and non pyrogenic, non-toxic, non-allergenic, non-immunogenic
- ✓ Totally absorbed in 3-4 weeks, leaves no residue and encapsulation

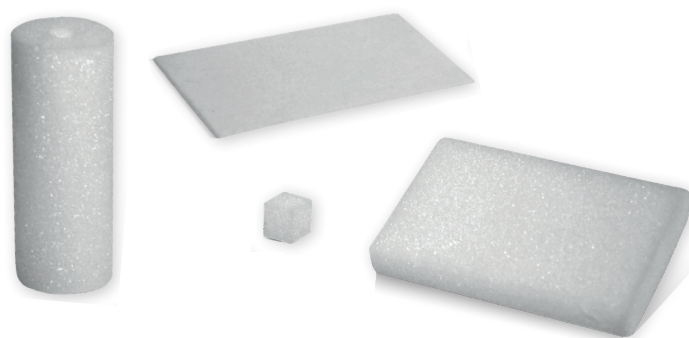


For quality
&
safety

smi
www.sutures.be

SMI-SPON

ABSORBABLE GELATIN SPONGE

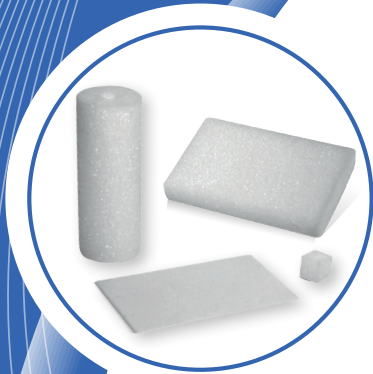

What is SMI-SPON

SMI-SPON is a surgical haemostatic sponge, manufactured from highly purified first grade gelatin material for use in various surgical procedures.

It is a quick and effective haemostatic which stops bleeding fast. It reduces intraoperative bleeding and surgery time.

Why using SMI-SPON

SMI-SPON can be used in various surgeries for haemostasis. When applied in vivo and used in appropriate amounts, it is completely absorbed within 3-4 weeks. SMI-SPON is biocompatible, non-toxic and non-pyrogenic.

SMI-SPON is gamma sterilized and individually packed in ready to use blister packaging.

Where to use

- Dental surgery
- Plastic surgery
- Vascular surgery
- Orthopaedic surgery
- ENT surgery
- Neuro surgery
- Abdominal surgery
- Anorectal surgery
- Genito Urinary surgery
- Gynaecological surgery

How to use SMI-SPON

- Cut a piece to the desired size
- Apply SMI-SPON either in a dry state or saturated with physiological salt solution
- If used dry, a piece of SMI-SPON is applied to the bleeding place and held in place with moderate pressure until hemostasis results
- If used wet, first immerse in the solution and then withdraw, squeeze between gloved fingers to expel air bubbles, and the replace in saline until needed

| Product | | Unit | Order no. |
|-----------------|-----------------|--------|------------------|
| Regular | 70 x 50 x 10 mm | 10 Pc. | ZHG705010 |
| Standard | 80 x 50 x 10 mm | 10 Pc. | ZHG805010 |
| Dental | 10 x 10 x 10 mm | 32 Pc. | ZHG101010 |
| Tampon | 80 x 30 mm DIA | 6 Pc. | ZHG8030 |
| Special | 80 x 50 x 1 mm | 20 Pc. | ZHG805001 |



For quality
&
safety

smi
www.sutures.be